

## **INSTRUCTION FOR USE**

### **Material Used:**

- P.P. (Polypropylene)
- A.B.S. (Acrylonitrile butadiene styrene)
- P.V.C. (Poly Vinyl Chloride).
- Master Batch (Blue)
- Master Batch (White)
- H.D.P.E. (High Density Polyethylene)
- Latex Bulb.
- Stainless Steel.

### **Indications:**

- Transfusion of blood and blood component.
- Maintaining blood requirement during surgery and excessive blood loss.

### **Contraindications:**

- Administration of highly viscous fluids.
- Do not use in patients with known hypersensitivity to any of the materials used.
- Do not use in Inflammation or infection of the insertion site.
- The device cannot be retained in human body for more than 24 hours.
- Don't use if package is opened or damaged.
- Don't administrate blood aspirated from contaminated or septic wound.
- Don't use with active seizure disorder patient.

### **Instruction for Use:**

#### ➤ **Before using sterile blood administration set:**

- Perform hand hygiene; this prevents the transmission of microorganisms.
- Wash hands with warm soapy water, Dry thoroughly then Put on gloves and aprons to reduce risk of infection.
- Apply clean gloves, to decrease the transfer of microorganisms.

#### ➤ **During using sterile blood administration set:**

- Close completely flow control clamp.
- Remove spike protector and insert full length of spike into the container.
- Suspend container & fill the drip chamber half full with blood by squeezing and releasing.
- Remove the needle protector. Open the flow control clamp gradually allowing the blood to displace air entirely in tubing and needle. Then close clamp tightly.
- Perform vein puncture & regulate the flow rate by opening the clamp gradually.

#### ➤ **After using sterile blood administration set:**

- Remove gloves and place patient in a comfortable position.
- Assess patient's level of comfort.
- Perform hand hygiene.

### **Cautions:**

- Do not use if package is open or damaged.
- Use rubber part only for injecting the medicines.
- For Single use.
- Discard after single use.

### **Conditions of Handling, Preservation and Storage:**

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature

**Warning:**

**Consequences Of Device Re-Use:**

- The device is Gram-negative bacteria even if the bacteria themselves are killed. for single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.



**Obelis<sup>SA</sup>**  
53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
[www.obelis.net](http://www.obelis.net)



Sterilized by Ethylene Oxide



Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date



Catalogue No.



Non-phthalate



Don't use if package is not intact



Non-Pyrogenic

**ISSUE/REV DATE:** 20-12-2020

**Assiut Factory:** Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

**& Fax:**002-088-4964222& **Mob:** 002-01001558853, 002-01068832355

**Cairo Head Office:** 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

**Assiut Office:** 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

**Alexandria Office:** 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988& **Mob:** 01223948666

**E-mail:** [ultra@elaggargroup.com](mailto:ultra@elaggargroup.com), [shady@elaggargroup.com](mailto:shady@elaggargroup.com) **Website:** [www.ultramedumic.com](http://www.ultramedumic.com)