



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE

Material Used:

- P.P. (Polypropylene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride).
- Poly Carbonate.
- L.D.P.E. (Low Density Polyethylene).

Indications:

- Infusion of fluid such as parenteral nutrition and administration of other drugs and medicines.
- Maintaining hydration and/or correct dehydration in patients who are unable to take sufficient volume of oral fluids.
- Male luer lock at one end & female luer lock at other end.
- High Pressure Extension Line.
- For pressure up to 54 bar (800psi).

Contraindications:

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

- Close completely flow control clamp of I.V. Infusion Set.
- Connect the Female luer connector of Extension Tube to the male fitting of Infusion set.
- Remove the protective cap of male lure connector and open roller clamp of I.V. Sets. Expel the air from the PVC tube and close roller clamp.
- Connect the male lure connector to the injection site (I.V. Cannula etc.) and regulate the flow by adjusting roller clamp to achieve desired flow rate.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.

EC REP

Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net

CE
2803

STERILE EO

Sterilized by Ethylene Oxide

LOT

Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date



Non-Pyrogenic

REF

Catalogue No.



Don't use if package is not intact



Non-phthalate

ISSUE/REV DATE: 20-12-2020



Assiut Factory: From Part No. 304 to Part No. 310 and Part No. 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt..
Tel: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax:002-088-4964222& Mob: 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis Tel: 022/4171621-4143794 & Fax: 022/4171613 & Mob: 01223988200 Assiut Office: 23, July Str. Tel: 088/2364111 – 2364222 & Fax: 088/2334964 & Mob. : 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran Tel: 03/5856202– 5856458 & Fax: 03/5828988& Mob: 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com Website: www.ultramedumic.com