



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE

Material Used:

- P.P. (Polypropylene)
- Stainless Steel different sizes.
- Silicon Tube
- Master Batch (colored).
- H.D.P.E. (High Density Polyethylene).
- L.D.P.E. (Low Density Polyethylene)

Intended Use

- The I.V. Cannula family is designed to provide access into the peripheral vascular system for administration of fluids and drugs.

Indications:

- Infusion of fluid such as parenteral nutrition and administration of other drugs and medicines.
- Maintaining hydration and/or correct dehydration in patients who are unable to take sufficient volume of oral fluids.

Contraindications:

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.
- Administration of irritant fluid or drugs.

Instruction for Use:

➤ Before using I.V. Cannula

- Explain to patient you are going to using I.V. Cannula for him, to reassure patient during procedure.
- Perform hand hygiene; this prevents the transmission of microorganisms.
- Wash hands with warm soapy water, Dry thoroughly then Put on gloves and aprons to reduce risk of infection.
- Apply clean gloves, to decreases the transfer of microorganisms.

➤ During using I.V. Cannula

- Select proper type and size of I.V. Cannula. Always use the smallest size cannula possible to adequately deliver to I.V. Fluids.
- Carefully select and clean the vein puncture site.
- Inspect cannula package for integrity and expiry and then remove cannula from package.
- Remove needle cover and grip the cannula from the injection port cap and needle hub projection.
- Insert cannula into the vein at a low angle and check for blood in flashback chamber to ensure proper vine puncture.
- Advance the catheter into vein and simultaneously withdraw the needle gently.
- Do not try reinsert a partially or completely withdrawn needle.
- Avoid blood spillage by pressing a finger on the vein, on or above the catheter tip.
- Replace the needle with either the I.V. Infusion Set line or the luer lock plug.
- Discard the needle in an appropriate waste container.
- Tape down the wings of the cannula on the patient's skin.
- Cover the puncture site with a sterile dressing.
- Drug can be injected intermittently with the help of a syringe through the injection port after removing the port cap.
- The injection port cap should remain when not in use.
- Inspect vein puncture site regularly for any reactions and check all connections.

➤ after using I.V. Cannula

- Remove gloves and place patient in a comfortable position.
- Assess patient's level of comfort.
- Perform hand hygiene.

Cautions:

- Read Instruction before use.
- This product should be used by a qualified Doctor or a Paramedic.
- Check the product and package integrity prior to use and also the expiry date.
- The product is sterile, non-pyrogenic and non-toxic if package is not opened or damaged.
- For Single use only. Discard after single use. Do not recycle, clean or resterile.
- Use the product immediately after opening the individual blister packing.
- Store in a cool & dry place. Protect from excessive heat or direct sunlight.
- The product should be used according to instruction for use.

ULTRA FOR MEDICAL PRODUCTS CO. DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 – 32 °C).



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



Lot No.



For Single Use Only



Latex Free



See Instruction for use



Date of Manufacturing



Use by / Expiry date



Catalogue No.



Non-Pyrogenic



Non-phthalate



Don't use if package is not intact



ISSUE/REV DATE: 20-12-2020

Assiut Factory: : Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt. **Tel:** 088/4960500 – 4960600 & **Fax:** 088/4960400 & **Mob:** 0101558853
Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 & **Fax:** 022/4171613 & **Mob.:** 0123988200
Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 & **Fax:** 088/2334964 & **Mob.:** 0123988202
Alexandria Office: 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 & **Fax:** 03/5828988& **Mob.:** 0123948666
E-mail: ultra@elaggargroup.com, shady@elaggargroup.com **Website:** www.ultramedumic.com