

**ULTRA  
MED**



شركة التت  
للمنتجات الطبية

**ULTRA FOR MEDICAL PRODUCTS CO.**

## **INSTRUCTION FOR USE**

### **Material Used:**

- Poly Carbonate.
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- Master Batch (White).
- Master Batch (Blue).

### **Indications:**

- Infusion of fluid such as parenteral nutrition and administration of other drugs and medicines.
- Maintaining hydration and/or correct dehydration in patients who are unable to take sufficient volume of oral fluids.
- Transfusion of blood or blood components.

### **Contraindications:**

- Administration of highly viscous fluids.
- Blood transfusion
- Not to be used in patients with known hypersensitivity to any of the materials used.

### **Instruction for Use:**

- Three Way Stop Cock is connected into intravenous (I.V. Cannula) for delivery of two fluids or drugs at a time.
- Inspect Three Way Stop Cock package for integrity and expiry and then remove from package.
- Remove Luer Lock Plug of I.V. Cannula and insert male part of Three Way Stop Cock into female part of I.V. Cannula.
- Connect I.V. Infusion set to both female parts of Three Way Stop Cock after removing Luer Lock Plug.
- The Three Way Stop Cock can be used to administer two drugs or fluids at the same time.

### **Warning:**

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

### **Cautions:**

- Read Instruction before use.
- This product should be used by a qualified Doctor or a Paramedic.
- Check the product and package integrity prior to use and also the expiry date.
- The product is sterile, non-pyrogenic and non-toxic if package is not opened or damaged.
- For Single use only. Discard after single use. Do not recycle, clean or resterile.
- Use the product immediately after opening the individual blister packing.
- Store in a cool & dry place. Protect from excessive heat or direct sunlight.
- The product should be used according to instruction for use.

ULTRA FOR MEDICAL PRODUCTS CO. DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

### **Conditions of Handling, Preservation and Storage:**

- Not more than 5 cartons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.

**ULTRA  
MED**



شركة التترا  
للمنتجات الطبية

**ULTRA FOR MEDICAL PRODUCTS CO.**

**EC REP**

**Obelis<sup>SA</sup>**

53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
[www.obelis.net](http://www.obelis.net)

**CE**  
2803

**STERILE EO**

Sterilized by Ethylene Oxide

**LOT** Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date

**REF**

Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



NON-DEHP

Non-phthalate

**ISSUE/REV DATE: 20 -12- 2020**



**Assiut Factory:** Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

**& Fax:**002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

**Cairo Head Office:** 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

**Assuit Office:** 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

**Alexandria Office:** 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

**E-mail:** [ultra@elaggargroup.com](mailto:ultra@elaggargroup.com), [shady@elaggargroup.com](mailto:shady@elaggargroup.com)

**Website:** [www.ultramedumic.com](http://www.ultramedumic.com)