

INSTRUCTION FOR USE

Material Used:

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride).
- P.V.C. (Poly Vinyl Chloride) sheets.

Indications:

- Used to collect the urine of the patient from the bladder in case of urinary retention.

Contraindications:

- Can cause infection to the patient.

Instruction for Use:

- Remove the Luer Lock cover of inlet connector from the tube and connect the tube with the urinary catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Collect the urine and put the Luer Lock cover after finishing.

Cautions:

- Discard after single use.
- Use only if the package is intact

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 – 32 °C).



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Non-Sterile



For Single Use Only



Latex Free



See Instruction for use



Lot No.



Date of Manufacturing



Use by / Expiry date



Catalogue No.

ISSUE/REV DATE: 01 September 2013

Assiut Factory: Industrial Area, Arab El Awamer, Abnoub, Assiut, Egypt. **Tel:** 088/4960500 – 4960600 & **Fax:** 088/4960400 & **Mob.:** 0101558853
Cairo Head Office: 64, Nakhla El Motaay Triumph Square Heliopolis **Tel:** 022/4171621-4143794 & **Fax:** 022/4171613 & **Mob.:** 0123988200
Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 & **Fax:** 088/2334964 & **Mob.:** 0123988202
Alexandria Office: 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 & **Fax:** 03/5828988 & **Mob.:** 0123948666
E-mail: ultra@elaggargroup.com, shady@elaggargroup.com **Website:** www.ultramedumic.com

Instruction For Use

ISSUE/REV: - 2/0

Page 1 of 1

URINE COLLECTION BAG EMPTY BAG