

INSTRUCTION FOR USE

Material Used:

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride).
- P.V.C. (Poly Vinyl Chloride) sheets.

Indications:

- Used to collect the urine of the patient from the bladder in case of urinary retention.

Contraindications:

- Can cause infection to the patient.

Instruction for Use:

- Remove the Luer Lock cover of inlet connector from the tube and connect the tube with the urinary catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Collect the urine and put the Luer Lock cover after finishing.

Cautions:

- Discard after single use.
- Use only if the package is intact

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 – 32 °C).



EC REP

SA

Obelis
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net

CE
2803

STERILE EO



Sterilized by EO

For Single Use Only



See Instruction for use



Non-Pyrogenic

LOT

Lot No.



Date of Manufacturing



Use by / Expiry date

REF

Catalogue No.



Don't use if package is not intact



Non-phthalate

- **ISSUE/REV DATE:** 20 December 2020

Instruction For Use

ISSUE/REV: - 2/0
Page 1 of 1

URINE COLLECTION BAG EMPTY BAG