

INSTRUCTION FOR USE

Material Used:

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride).
- P.V.C. (Poly Vinyl Chloride) sheets.

Indications:

- Used to collect the urine of the patient from the bladder in case of urinary retention through Nelaton or Foley's catheter and to measure the volume of urine for daily checks.

Contraindications:

- Can cause infection to the patient.

Instruction for Use:

- Remove the cover of inlet connector from the tube and connect the tube with the Foley's or Nelaton catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Drain the urine using outlet connector if the bag is filled more than half of the capacity.

Cautions:

- Discard after single use.
- Use only if the package is intact

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 – 32 °C).



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide

LOT Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date



Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



Non-phthalate

- **ISSUE/REV DATE:** 20 December 2020

