

Yankaur Suction Unit

Code No.: 9008

1- Description:

- Suitable for pre-operative removal of secretion and body fluids.



2 - Advantages & Features:

- Standard funnel connector to suit different suction pump lines at proximal end.
- Standard funnel connector at distal end of tubing to connect Yankaur Handle.
- Yankaur Handle is open end with 4 lateral eyes to withstand negative pressure and ensures continuous and easy suction of fluid.
- Soft and kink resistant ribbed PVC tubing.
- Available in Standard Tip Handle & Crown Tip Handle.

3- Indications:

- The Yankaur suction tube is used to clean up potentially infectious secretions and debris on the tube after aspiration, leaving it drier and cleaner between uses.

4- Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

5- The purpose:

- Suitable for pre-operative removal of secretion and body fluids.

6- Instruction for Use:

- Fix small connector on handle and large connector at the other end on suction pump.
- Close vacuum control window by placing thumb on it.

ULTRA FOR MEDICAL PRODUCTS CO.

- Insert tip of handle in operative body part and let it remove secretion by sucking.
- Vacuum can be controlled on regular intervals by vacuum control window.

7- Care instructions:

- Do not use if package is open or damaged.
 - Discard after single use.
- The product should be used according to instruction for use.

8- Storage conditions:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 –32 OC).

9- Product code with size and measures:

- Code No.: 9008.
- Tube die: I Ø 6.0 mm, O Ø 8.5 mm.
- Tube length 210 cm, 250 cm and 300 cm.

Code	Type
9008-01	Yankaur Handel
9008-02	Yankaur Suction Set (Crown Tip)
9008-03	Yankaur Tube

10- List of product components:

PVC tubing - Yankaur Handel - Funnel
connector -

11- Sterility:

- Sterilized by Ethylene oxide Gas (20% ETO +80 % Carbone dioxide) .

12-Shelf life: 5 Years.

Assiut Factory: Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)
& Fax:002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com