



**ULTRA FOR MEDICAL PRODUCTS CO.**

## **INSTRUCTION FOR USE**

### **Material Used:**

- P.P. (Polypropylene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride).
- Master Batch (Blue)
- Master Batch (Red)
- H.D.P.E. (High Density Polyethylene)
- Stainless Steel

### **Indications:**

- Transfusion of blood and blood component.
- Maintaining blood requirement during surgery and excessive blood loss.

### **Contraindications:**

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.

### **Instruction for Use:**

- Connect the Female Luer Connector of Fistula Needle to the male fitting of transfusion set.
- Remove the protective cap of needle of Fistula Needle and open the roller clamp of Transfusion set.
- Expel the air from the PVC tube and close the roller clamp or click clamp.
- Insert the Needle into the vein and regulate the flow by adjusting the roller clamp to achieve desired flow rate.

### **Warning:**

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

### **Cautions:**

- Do not use if package is open or damaged.
- For Single use.
- Discard after single use.

### **Conditions of Handling, Preservation and Storage:**

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature

**EC REP**

**Obelis<sup>SA</sup>**  
53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
[www.obelis.net](http://www.obelis.net)

**CE**  
2803

**STERILE EO**

Sterilized by Ethylene Oxide



For Single Use Only



See Instruction for use

**LOT**

Lot No.



Date of Manufacturing



Use by / Expiry date

**REF**

Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



**ISSUE/REV DATE:** 20-12-2020

**Assiut Factory:** Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

**& Fax:**002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

**Cairo Head Office:** 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

**Assuit Office:** 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

**Alexandria Office:** 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

**E-mail:** [ultra@elaggargroup.com](mailto:ultra@elaggargroup.com), [shady@elaggargroup.com](mailto:shady@elaggargroup.com) **Website:** [www.ultramedumic.com](http://www.ultramedumic.com)