

INSTRUCTION FOR USE

Material Used:

- ABS(Acrylonitrile Butadiene Styrene)

Indications:

- used to clamp the umbilical cord immediately after the birth of a child

Instruction for Use:

Steps to Cut the Umbilical Cord

1. Be sure the cord has stopped pulsing for most births. This is called delayed cord clamping and provides benefits for your baby.
2. Ensure that there are two clamps on the cord. (The practitioner will be responsible for this part.)
3. Hold the section of cord to be cut with a piece of gauze under it.
4. Using sterile scissors cut between the two clamps.
5. Dab excess blood. (The amount of blood depends on how long you wait to cut the cord, the longer you wait, the less blood.)
6. Place scissors away or hand them back to whoever handed them to you.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Read Instruction before use.
- This product should be used by a qualified Doctor or a Paramedic.
- Check the product and package integrity prior to use and also the expiry date.
- The product is sterile, non-pyrogenic and non-toxic if package is not opened or damaged.
- For Single use only. Discard after single use. Do not recycle, clean or resterile.
- Use the product immediately after opening the individual blister packing.
- Store in a cool & dry place. Protect from excessive heat or direct sunlight.
- The product should be used according to instruction for use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date



Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



Non-phthalate

ISSUE/REV DATE: 20-12-2020

Assiut Factory: Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax:002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Aired Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com